



Using Prefilled Luer Lock Syringes

Preassembled with the UltraSafe Passive®[†] Delivery System

System Components

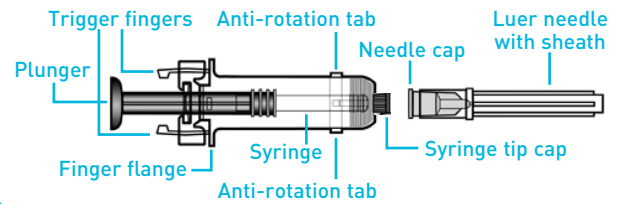


1

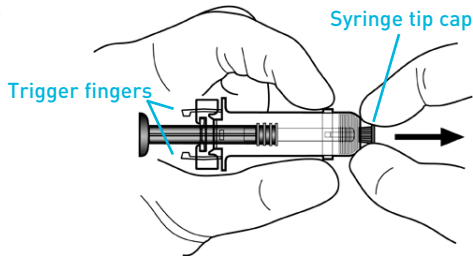

- At any of the following steps, avoid contact with the trigger fingers to keep from activating safety device prematurely.
- Thoroughly agitate the syringe to maintain suspension of the vaccine. Then, inspect the vaccine. Refer to the Prescribing Information for vaccine's proper appearance.
- Remove the syringe tip cap and the cap from the needle (supplied).*


*If a different needle is chosen, it should fit securely on the syringe and be no longer than 1 inch to ensure proper functioning of the safety device.

2

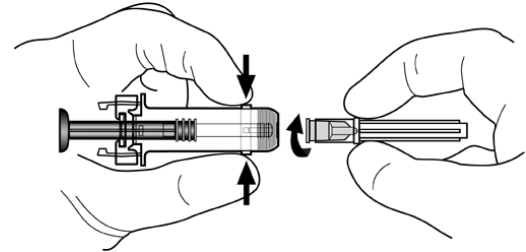


- Insert the needle (supplied).
- Press the 2 anti-rotation tabs near the tip of the syringe and twist the needle clockwise to secure it completely to the syringe.

3

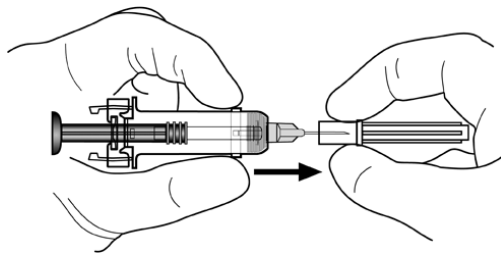


- Remove the sheath of the needle supplied.

4

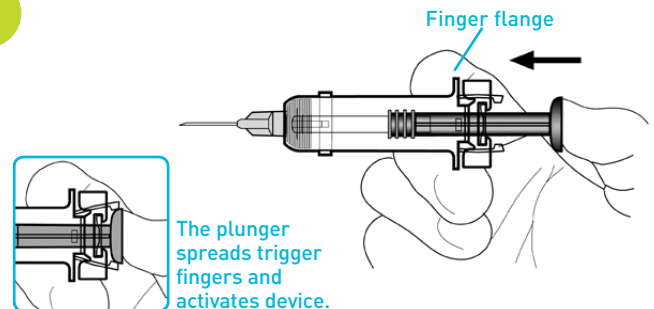

- Place your fingers on the finger flanges behind the needle. Administer injection using standard protocol.
- Depress the plunger until entire dose has been given. The needle guard will **not** activate unless the **entire** dose has been given.

5

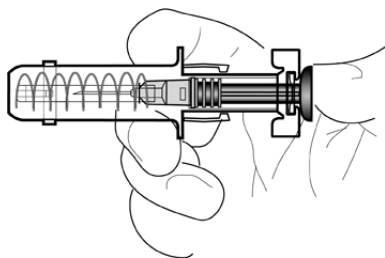

- With the plunger still depressed, withdraw the needle from the patient.
- After withdrawing the needle from the patient, **slowly** release the plunger.
- The needle guard moves into position to cover the entire needle.

6

After the injection

- Peel off the labels and attach them to your patient's chart for documentation of vaccination.
- Dispose of the syringe in the approved sharps container.

To order prefilled syringes

- Contact your Merck Vaccine Representative, or
- Visit us on the Web at MerckVaccines.com®.

Please see instructions for use in Dosage and Administration section of appropriate product Prescribing Information.

[†]UltraSafe Passive® delivery system is a registered trademark of Safety Syringes, Inc.



MerckVaccines.com is a registered trademark of Merck & Co., Inc.
Copyright © 2008 Merck & Co., Inc. All rights reserved.

20852158(1)-08/08-HBV
Printed in USA
Minimum 10% Recycled Paper ♻️